

Product Information


Nail


REF NO.	Product Description	Specification
070260110	Universal Humeral Nail, Cannulated	φ7.0x110mm
070260160	Universal Humeral Nail, Cannulated	φ7.0x160mm
070260180	Universal Humeral Nail, Cannulated	φ7.0x180mm
070260200	Universal Humeral Nail, Cannulated	φ7.0x200mm
070260220	Universal Humeral Nail, Cannulated	φ7.0x220mm
070260240	Universal Humeral Nail, Cannulated	φ7.0x240mm
070260260	Universal Humeral Nail, Cannulated	φ7.0x260mm
070270110	Universal Humeral Nail, Cannulated	φ8.0x110mm
070270160	Universal Humeral Nail, Cannulated	φ8.0x160mm
070270180	Universal Humeral Nail, Cannulated	φ8.0x180mm
070270200	Universal Humeral Nail, Cannulated	φ8.0x200mm
070270220	Universal Humeral Nail, Cannulated	φ8.0x220mm
070270240	Universal Humeral Nail, Cannulated	φ8.0x240mm
070270260	Universal Humeral Nail, Cannulated	φ8.0x260mm

Others


REF NO.	Product Description	Length (mm)
070280018- 070280050	3.6mm Locking Screw	18-50
070290020- 070290050	5.0mm Locking Screw	20-50
070300009	End Cap	

Universal Humeral Nail

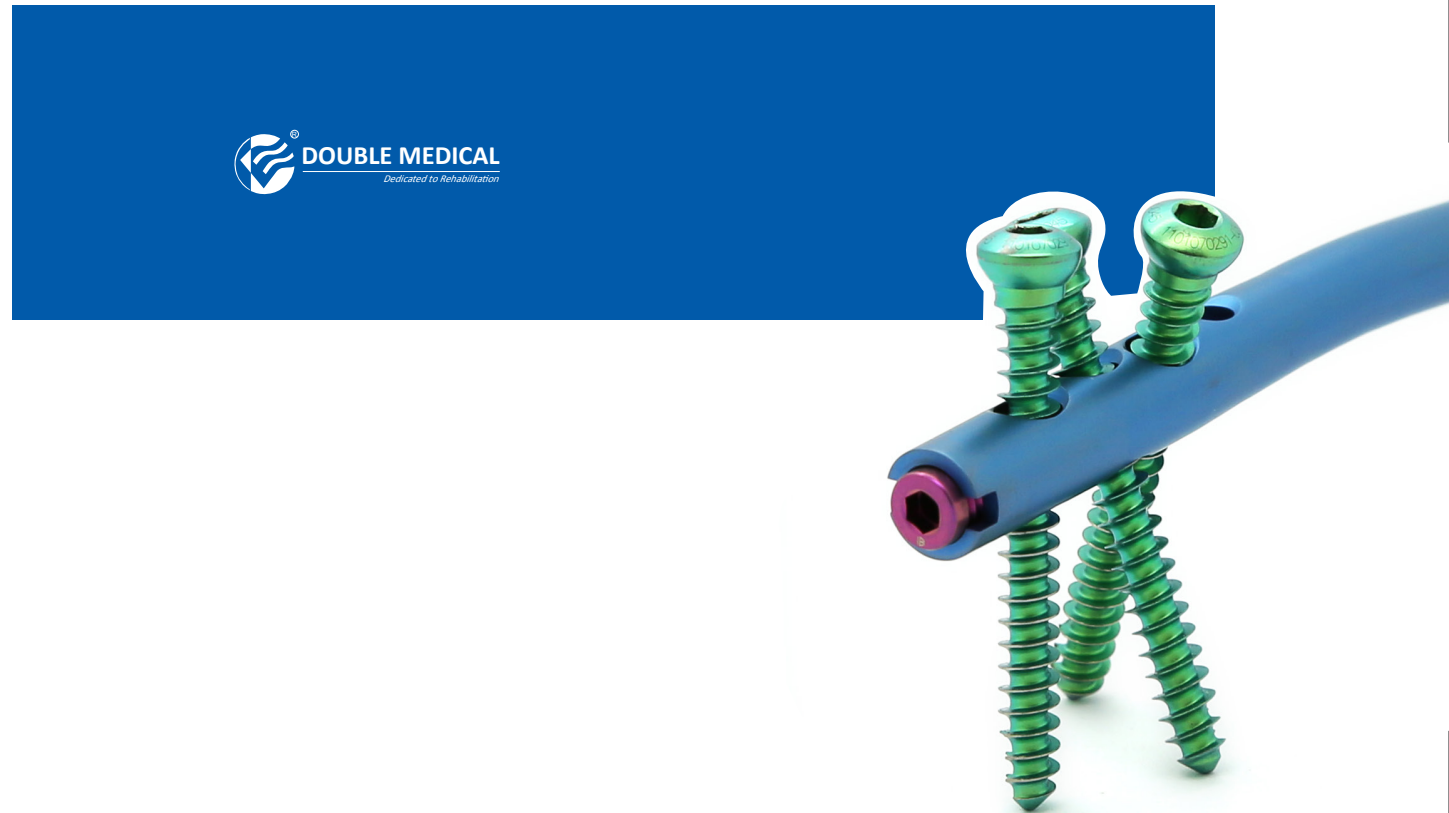


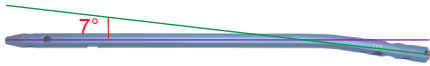


The angle is 7° which allows the insertion entry point on the great tubercle to avoid damaging axillary nerve.

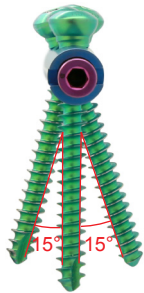


3pcs of 5.0mm Proximal Cancellous Screws provide bone holding purchase and angular locking stability for multi-planar fixation of fractures.

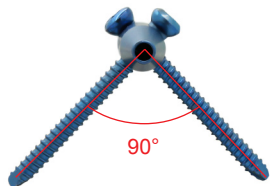


90° cross locking provides higher angular stability in the distal.

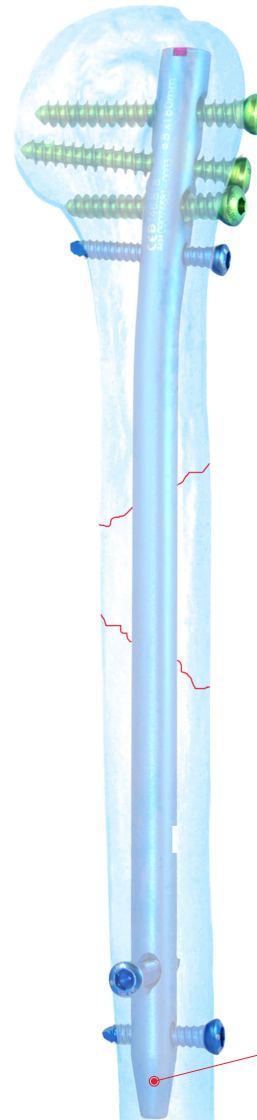

Anatomic design corresponds to the physiological curvature of the medullary cavity for easier insertion.

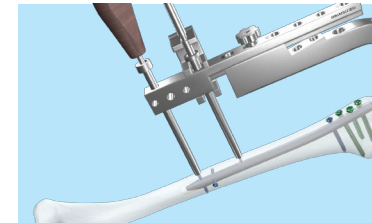

Distal aiming device eases the surgical operation for insertion of distal locking screws.

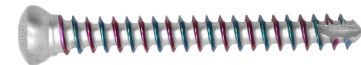

All locking screws designed with double lead thread for easier insertion.

Distal cutting end makes the insertion easier.

Indications

- Fracture of humerus
- ### Contraindications
- General or local infection, osteomyelitis
 - Patient can't tolerate operation or anesthesia because of poor health
 - ORIF will lead to local infection or skin necrosis because of poor local soft tissue
 - Severe osteoporosis
 - Open fracture, obvious contamination

Warning

This flyer is just for understanding the specific product features. For clinical usage, please refer to the surgical guide. Instruction by experienced surgeon is highly recommended.



3.6mm Locking Screw



5.0mm Locking Screw